

Scientific Solutions for Fitness.

# **Users' Operations Manual**





Before using this product, read this manual and follow all safety rules and operating instructions.



This page purposely left blank

## TABLE OF CONTENTS

SAFETY INFORMATION	
INTRODUCTION	
SPECIFICATIONS	
SETUP & INSTALLATION	
PRODUCT OVERVIEW	
KEYPAD FUNCTIONS 6	
BIOMECHANICAL POSITIONING	
ADJUSTMENTS	
ACCESSORIES	
HEART RATE TRANSMITTER STRAP & CONTACT GRIPS	
PROGRAM INSTRUCTIONS11 -	· 19
MAINTENANCE & SERVICE 20	
WARRANTY	- 25

Order online 24/7 at www.SCIFIT.com

## SAFETY INFORMATION

The purpose of safety symbols is to attract your attention to possible dangers. The safety symbols, and their explanations, deserve your careful attention and understanding. The safety warnings do not, by themselves, eliminate any danger. The instructions or warnings they give are not substitutes for proper accident prevention measures.

Look for these symbols to point out important safety precautions. It means attention! Your safety is involved.

## A WARNING

Failure to obey a safety warning **can** result in injury to yourself and others. Always follow the safety precautions to reduce the risk of personal injury.

## 

Failure to obey a safety caution **can** result in injury to yourself and others. Always follow the safety precautions to reduce the risk of personal injury.

Read all instructions before using your SCIFIT exercise machine.Save these instructions for future reference.

Close supervision is necessary when exercise machine is being used by, or near children, or individuals with disabilities. Keep children away from extended hand and foot supports.

Use the exercise machine only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.

Never drop or insert any object into any opening.

Do not use outdoors or in a harsh environment where water is present (i.e. pool or spa areas).

1

## INTRODUCTION

Thank you for your purchase of the SCIFIT ISO1000R or ISO7000R Adjustable Position Recumbent Bike. We have incorporated the finest technology and ergonomic design into this machine to assist you in achieving your fitness goals. However, for your safety, please adhere to the following recommendations before you begin to exercise.

The SCIFIT ISO1000R & ISO7000R are designed for Class "S" (Studio), Class "I" (Special Needs) and Class "II a" (Medical Device Directive) applications. The intended uses are for Cardio & Pulmonary Rehabilitation as well as Sports Medicine, Physical Therapy, Physical Conditioning and Strength Training. SCIFIT equipment is suitable for placement in fitness, medical rehab, sports performance and recreational facilities. The ISO1000R & ISO7000R have 200 levels of computer controlled resistance, which ensures that users of all fitness levels will be able to get a challenging but not overwhelming workout.

#### **Consult Your Physician**

Consult your physician or medical specialist before participating in any exercise program, especially if you are pregnant, or if you are suffering from: heart disease, respiratory disease, diabetes, hypertension, high blood pressure, elevated cholesterol, arthritis, or any other diseases or physical complaints.

#### **Adult Usage**

SCIFIT equipment is designed and intended for adult usage and children of appropriate ages while being supervised. Young children should be kept a safe distance away from equipment.

#### **Cool Down**

To prevent muscle injuries and soreness, you should always cool down (at least 5 minutes) by doing a series of stretches after each workout.

### 

Always stretch after a workout session to reduce the risk of personal injury. Failure to stretch properly **may** result in injury to youerself.

#### **Guidelines for Healthy Adults**

The American College of Sports Medicine (ACSM) and the American Heart Association (AHA) are continuously updating guidelines and recommendations for physical activity.

The current basic recommendations from ACSM and AHA is:

Do moderately intense cardio 30 minutes a day, five days a week.

#### OR

Do vigorously intense cardio 20 minutes a day, three days a week.

#### AND

Do eight to ten strength-training exercises, eight to twelve repetitions of each exercise twice a week.

Moderate-intensity physical activity means working hard enough to raise your heart rate and break a sweat, yet still being able to carry on a conversation. It should be noted that to lose weight or maintain weight loss, 60 to 90 minutes of physical activity may be necessary. The 30 minute recommendation is

for the average healthy adult to maintain health and reduce the risk for chronic disease.

To review the latest guidelines and recommendations, visit the American College of Sports Medicine website: *www.acsm.org* 

### 

Stop exercising immediately if you feel nausea, dizziness, sharp pain, or any other physical discomfort. Failure to stop excercising **can** result in injury or death to yourself.

### **Train Intelligently**

To ensure a future of good health, you should always eat well balanced meals, drink plenty of fluid/water during a workout, and stay fit by exercising intelligently. Always follow training guidelines that are specific to your own physical condition.

### 

Injuries to health may result from incorrect or excessive training. Failure to stop excercising **can** result in injury or death to yourself.

## SPECIFICATIONS

Weight	<ul> <li>ISO1000R - 234 lbs. / 106 Kg</li></ul>
Accuracy Class Warranty	Class A Within U.S. & Canada 3 years parts, 1 year labor Outside U.S. 3 years parts, no labor
	58" 1473 mm
27" 686 mm	
55" 1397 mm	

## SETUP & INSTALLATION

### Placement of Cardiovascular Machine

Position the unit in a location that has a stable and level sub-floor. Equipment should be postioned in a location that is not in direct sunlight, in areas of extreme temperature and humidity, or where the equipment may be splashed with water or fluids. This machine is intended for indoor use only.


Equipment should be positioned with a safety perimeter of 40" or 1 meter around the unit.


## A WARNING

Failure to place the equipment on a stable and level sub-floor with a safety perimeter can result in injury to yourself and others.

### Level Cardiovascular Machine

If the sub-floor is unlevel, the unit is equipped with leveling end caps. The end caps are designed with the recess of end cap off centered. This allows you to rotate the endcap(s) to a position that will level the equipment. After adjustment, check and make sure end caps are fully seated against base frame tubes.

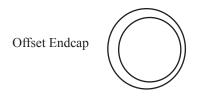

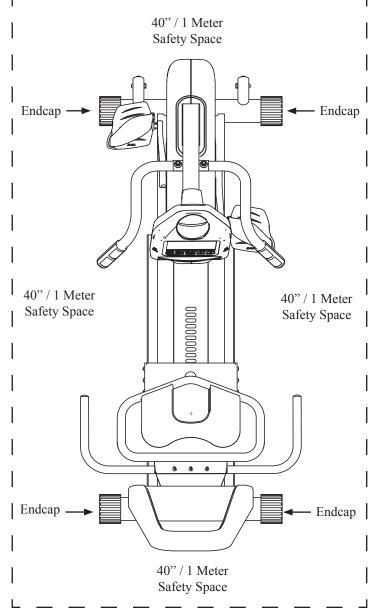


### **Proper Supervision**

Close supervision is required when the exercise machine is being used by or near individuals with disabilities. Keep unsupervised children outside the safety perimeter.

### Cleaning

Never clean the machine with organic solvents alcohol or bleach based cleaners. To promote a healthy enviroment, wipe any sweat off of the unit with a commercially manufactured wipe designed for the fitness industry, or a damp cloth.


## PRODUCT OVERVIEW

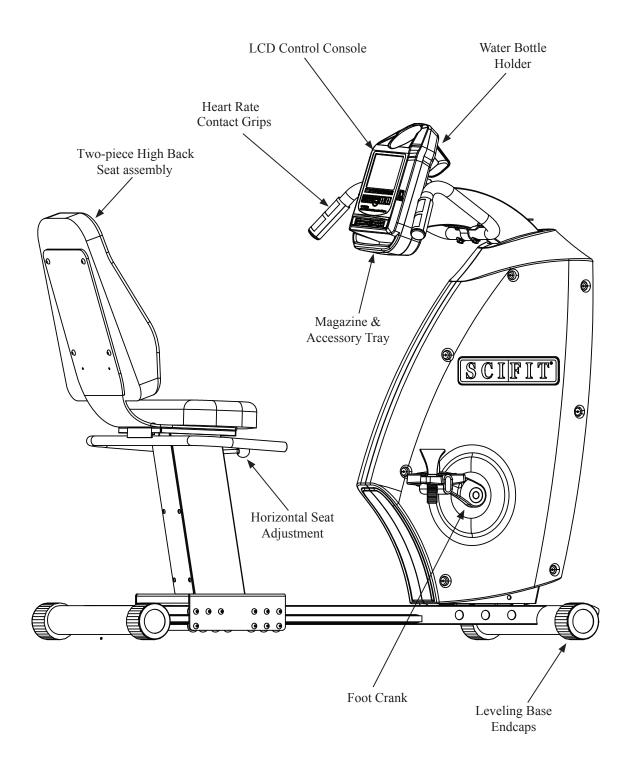

## **KEYPAD FUNCTIONS**

#### Work Level Status LCD Screen:

The Work Level Status LCD Screen gives a progressive real-time indication of the current work level of any and all programs

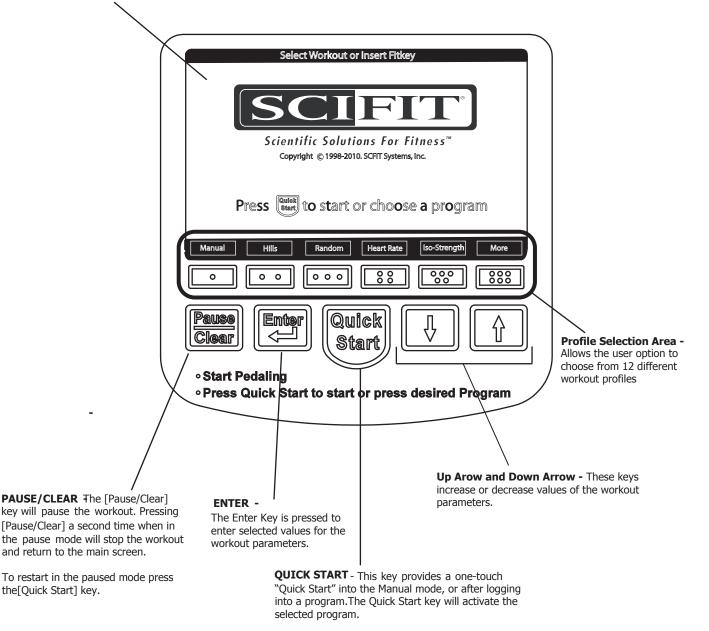

#### **Body Positioning**

Comfort is a key issue to obtain the most beneficial exercise. The seat should be positioned so that the legs are not hyperextended and the knee has a slight bend when the foot pedal is at the bottom of the stroke.

#### **Gripping the Handlebar**

Position the seat so that your grip on the handlebar is light and comfortable. The SCIFIT bike has two handlebars (center console and seat) that allows several gripping positions. Moving hand positions during a workout will help reduce stress points and upper body tension.

**Important Note:** When using this product for medical or rehabilitation purpose, the program selection, work load and seat position should be set by the attending therapist or medical professional based on the specific needs of the user.

### **A** CAUTION


Always follow medical professionals guidelines to reduce the risk of personal injury. Failure to follow their directions **may** result in injury to yourself.

## ADJUSTMENTS

### **Fixed Seat Adjustment**

The fixed seat may be adjusted horizontally in 1" increments to accomodate users of different heights and to obtain the proper position for leg extension.

To adjust the seat position horizontally, pull upwards on on the yellow knob and move the seat forward or back to comfortable position.

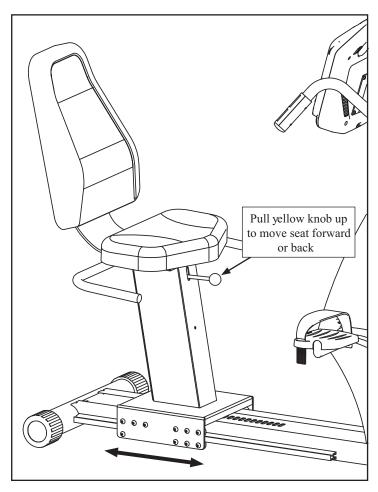

### **Swivel Seat Adjustments**


Horizontal Adjustment - The seat can be adjusted in 1" increments and is held in place with a spring loaded ajustment pin. The seat system is easily adjusted by pulling upward on the yellow T-handle and sliding the seat assembly forward or backward.

The seat assembly may also be adjusted by using the foot assist pedal located on the base.

Adjust the seat so the legs have a slight bend at full extension.

Vertical Adjustment - Pull upwards on the paddle shaped handle to move the seat up or down. **Note: No weight should be on the seat to adjust upward** 


Swivel Adjustment - Push downward on the paddle shaped handle to swivel right or left until it locks in place. The seat will lock into place on every 90 degree position.

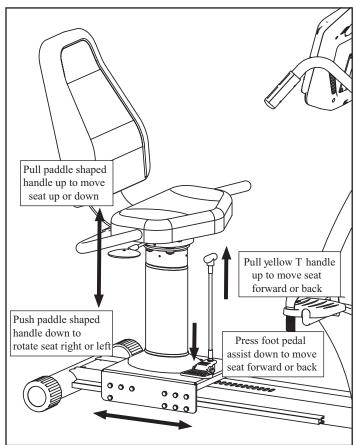

## ACCESSORIES


### 12V Power Pack - Dom. (#P1562) Int'l (#P3733)

To provide continuous power to the console when the machine is not in operation, plug in the 12V power pack into the jack located on the lower part of the right cover near the front of the machine.

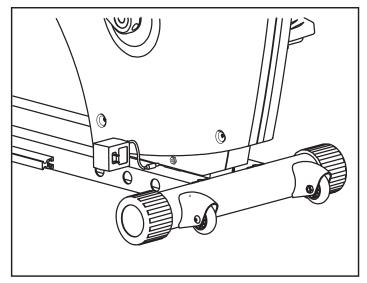

#### Adjustable Cranks - #P4313

The ISO1000R/7000R may also be fitted with adjustable foot cranks to allow the pedal to be set at different comfortable distances for the operator.

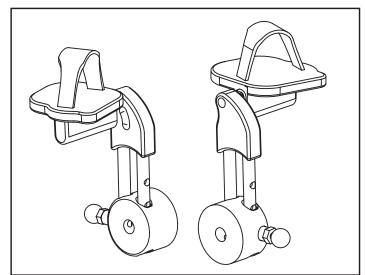

Order online at www.SCFIT.com

## HEART RATE TRANSMITTER STRAP & CONTACT GRIPS

#### Uses of the Heart Rate Transmitter Strap

The optional wireless heart rate transmitter accessory is a useful device if you desire continuous feedback on your current heart rate during a workout. In addition, the ISO1000R & ISO7000R include a Heart Rate Control program which requires use of the strap.


The Heart Rate Control program enables the user to setup a target heart rate. Through real-time monitoring of the user's heart rate, the computer maintains the user's heart rate at or below this established target rate by decreasing the workload if the user's heart rate exceeds the target.

#### **Activating & Deactivating Transmitter**

Heart Rate Transmitter Straps come in a variety of styles and manufacturers. To properly activate, secure strap to the chest and to deactivate follow the instructions that come with the Transmitter Strap.

## 

Always follow manufacturers instructions for proper care and use. Failure to follow manufacturers instructions may result in injury to yourself.

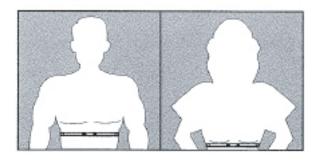

## A WARNING


Heart Rate monitoring systems may be inaccurate. If you feel faint, stop exercising immediately. Over exercising may result in serious injury or death.

#### **Use of the Heart Rate Contact Grips**

For those models with contact heart rate grips, simply grip the contact pads lightly. After a short delay the pulse reading will appear on the display. If a heart rate transmitter strap is worn, it will override the contact pickup.

The contact grips are not designed for use with SCIFIT's Heart Rate Control program. This program requires a steady, continuous telemetry signal which can not be provided using contract heart rate grips.

While the contact grips provide a good barometer for monitoring the users heart rate, they are not recommended for medical where accuracy of monitoring is vital.

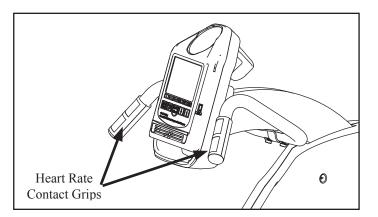

#### **Calculating Target Heart Rate Zone**

Your approximate **Maximum Heart Rate (MHR)** is equal to 220 minus your age. The upper limit of your target zone is equal to 0.85 times your MHR. The lower limit of your target zone is equal to 0.60 times your MHR.

For Example: Your age is 40: 220 - 40 = 180 180 x .85 = 153 (max MHR) 180 x .65 = 108 (min MHR)

These values are based upon averages, always consult your physician to establish the proper Heart Rate Zone for your individual health condition.

### Programs

SCIFIT products have 200 levels of computer controlled resistance, which ensures that users of all fitness levels will be able to get a challenging but not overwhelming workout. Because the resistance is adjustable in .1 increments, it is easy for the user to track progress. SCIFIT products feature one of the highest resistance capabilities available on any cardio equipment.

SCIFIT products are self generated and have an extremely low starting resistance. One pedal movement or crank revolution activates the console and the screen will remain lit at the most minimal levels of movement.


SCIFIT products have a battery back up which will keep the screen lit for a short period after exercise or if the user stops. This varies between 0 seconds and 5 minutes and can change with the amount of use each product gets.


If you prefer the console to remain lit before and after use, an optional wall pack can be plugged in to accomplish this. This is common in medical settings and with Fit-Key Programs.

### Manual Workout

Allows the user to set the intensity at any level. This level will not change unless the user manually increases or decreases the intensity using the [UP] or [DOWN] arrow keys.



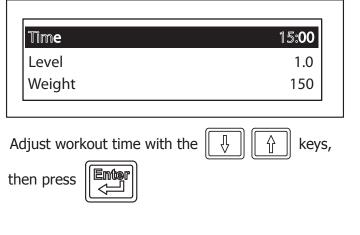

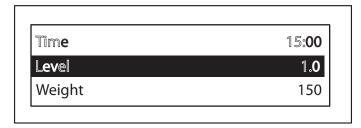


Adjust level intensity with the

Start



keys, then press



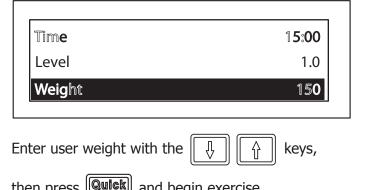

and begin exercise.

### **Fit Quik**

then press

Allows the user to select a two stage program of cardiovascular work with ISO-strength (Isokinetic) work loads and duration determined within the factory set up.



11

**Constant Work** 

given setting.

Pedal resistance is automatically adjusted to

compensate for any change in pedal speed (RPM),

thus workload (Watts) will remain constant at any

### Fit-Quik Cont'd

The categories displayed will depend on the workout that is selected in the factory setting. The following will be seen for the workout choosen.

Manual or Hills


Level       1         Weight       150         Adjust the Level and Weight with the       Image: Compare the second	Press <b>CONSTANT WORK</b> Then
Constant Workout	Adjust workout time with the  🗍
Adjust the Weight with the	Time15:00Watts20Weight150Adjust target watts with theImage: Descent sector sec
	keys, then press
	Enter the user weight with the keys, then press Start and begin exercise
	12

### **ISO-Strength Program**

ISO-Strength is a isokenitic program, meaning that the unit will resist your movements in order to force you to stay at a set speed. This type of strength workout will not cause excessive soreness because it is accommodating. The computer will adjust resistance to keep the user at the target RPM speed.



Time	1 <b>5:00</b>
RPM	35
Weight	150
Adjust workout time with the 🗍	
keys, then press	
Time	15:00
RPM	35
Weight	150
Adjust target RPM with the 🔛 [	ੀ keys,

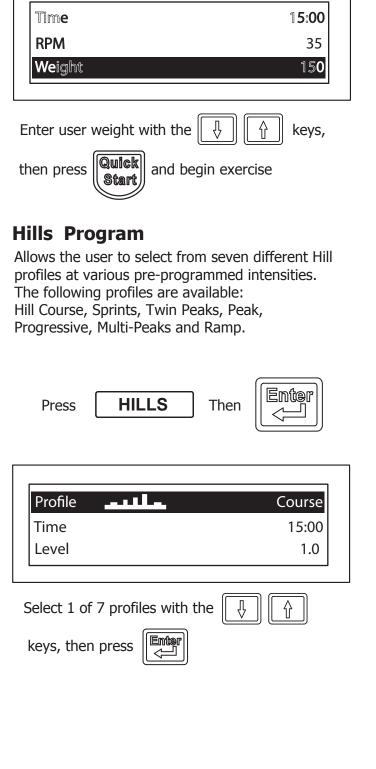

### Hills Cont'd

ProfileCourseTime15:00Level1.0	of t scre pre
Adjust workout time with the 🕂 🟠	An Hea Hea <b>He</b> a
Profile Course Time 15:00 Level 1.0	The pro inst hov
Adjust level intensity with the 🗍 🗍 keys, then press	
Time         15:00           Level         1.0           Weight         150	Pi
Enter user weight with the then press Start and begin exercise	
	Adj key <b>14</b>
SCIFIT • Scientific Solutions for Fitness •	

### MORE

Pressing this key will enter the user into the 2nd of three workout program selection screens. This screen will show an additional five more pre-programmed workout profiles.


Press MORE

A new screen will appear with additional programs: Heart Rate, Stess Test, Random, Power Fit and Heart Fit.

### Heart Rate Program

The Heart Rate Transmitter Strap is required for this program. Refer to page 10 for a complete instruction on using the strap and to understand how to calculate your Target Heart Rate.

### • Put on Heart Rate Transmitter Strap

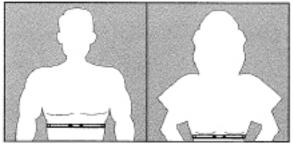

Press HEART RATE Then


Enter	

Time	15:00
Heart Rate	120
Weight Rate	150

Adjust the workout time with the  $\|$   $\{$ 

eys, then press

### Heart rate Cont'd

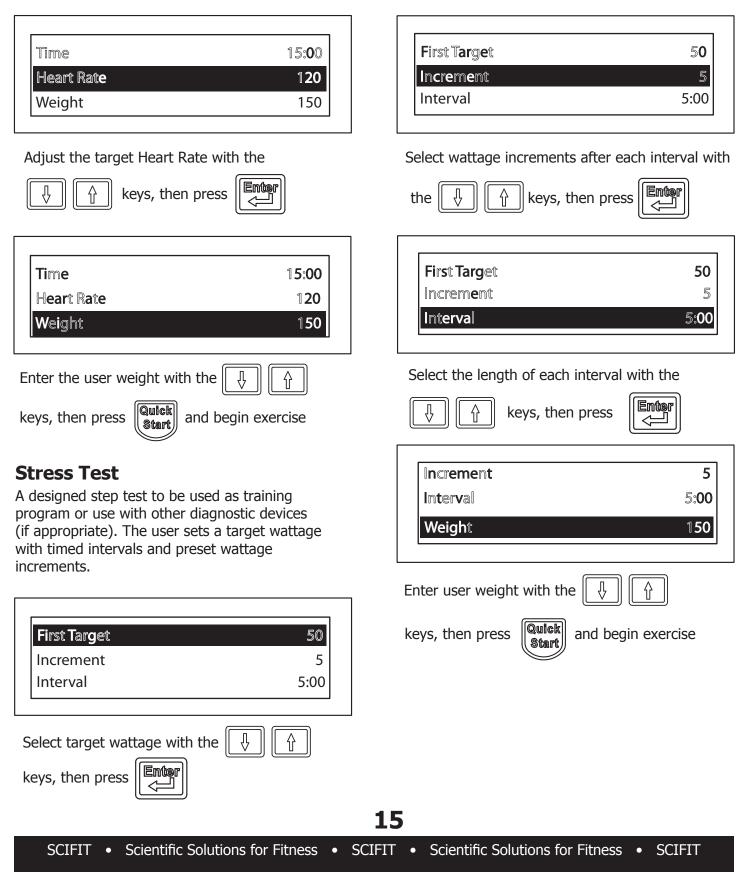

### **Random Program**

The Intelli-Fit console will randomly display different workout profiles each with its own level of difficulty. The operator only has to choose the type of the profile, then select the time length of the workout and the level difficulty.

Note: The highest column of the profile represents the workout level you selected. Columns below it are percentages of the level selected.

Press	RANDOM	Then	

Profile	Random
Time	15:00
Level	1.0
Select a random profiles with the	


keys, then press

Inter	

Profile	-	Random
Tim <b>e</b> Level		15:00 1.0

Adjust workout time with the

keys, then press

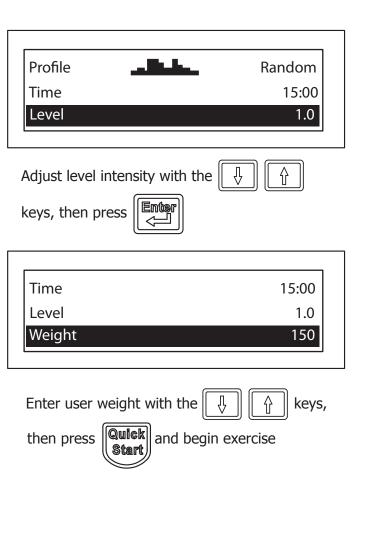


### **Power Fit Programs**

This key displays two options "Test" and "Training". These programs are power and strength indicators for the user.

Press

**Power Fit** and the screen will display

a choice of "Test" or "Training" as shown below.

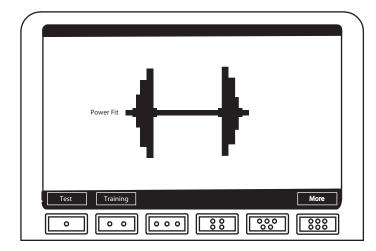

### Test

This program is based on a pre-programmed wourkout designed to evaluate your "Power Drop %", which gives a fitness indication of power and strength using wattage as the main measurement.



and begin exercise

### Training

Specifically designed preset wattage training programs based off user Power Fit test evaluation. Three programs and four levels (Beginner, Intermediate, Advanced and Sport Performance) are available.

Press <b>TRAINING</b> Then	er 
Program Level B Weight	1 Beginner 150
Select 1 of 3 programs with the keys, then press	
Program Level E Weight	1 Seginner 150
Select 1 of 4 levels with the	keys,
Program Level E Weight	1 8 <b>eginne</b> r 1 <b>50</b>
Enter user weight with the then press Start and begin exercise	keys,
7	

1


### Heart Fit Programs (Use Chest Strap)

This key displays two options "Test" and "Training". These programs are Heart Rate recovery indicators for the user.

Press

Heart Fit and the screen will display

a choice of "Test" or "Training" as shown below.

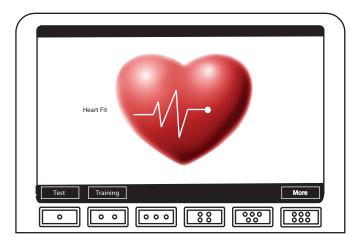

### Test

This program is based on a pre-programmed workout designed to evaluate your Heart Rate recovery. The user can select from 1 to 4 levels: Beginner, Intermediate, Advanced and Sports Performance.

Press TEST Then	Select 1 of 4 level with the 🐺 🟠 keys then press 📰
Level Beginner Weight 150	Level Beginner Program 1 Weight 150
Select 1 of 4 levels with the keys, then press	Select 1 of 3 programs with the 🐺 🗍
SCIFIT • Scientific Solutions for Fitness • SCIFIT	• Scientific Solutions for Fitness • SCIFIT

Leve Beginner Weight 150 Ŷ Enter user weight with the 

keys,

Beginner

1

150

then press Quick and begin exercise Start

## Training

Level


Program

Weight

Designed training programs based on the user's evaluated Heart Fit test. A training schedule with retesting is recommended. The user has three programs and four levels: Beginner, Intermediate, Advanced and Sports Performance to choose from.



### **Training Cont'd**

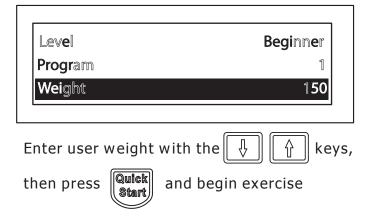

### **Additional Key Features**

SCIFIT has also included extra user friendly key features the user may use during a workout.

CHANGE VIEW

Allows the user to change the viewing fomat of the workout profile and data displayed on the LCD screen during the workout.

FAN

The console contains two 3-speed cooling fans that can be activated by the user at anytime.



When activated a count down will display on the screen for the user to prepare to pedal faster for a small duration of time. This time duration can be increased or decreased in the "User Set-up".



If at anytime the user may want to pause during a workout, this may done by pressing the [Pause/Clear] key. The screen workout will then minimize and provide the user three options

- 1. **Continue** Maximizes the workout screen and continues the workout where the user had stopped.
- 2. **Change** Returns to a sub-menu where information may changed that was previously entered by the user.
- 3. **Stop** Exits the workout and returns to the main menu.
- Note: The console will reset to the main menu after a short period of time if no selection is made.

### Maintenance

The ISO1000R & ISO7000R are virtually maintenance free. After training, always wipe down your SCIFIT exercise product. Perspiration that continuously settles on the frame, upholstery, casings, and control console may eventually cause rust or damage. Damage resulting from lack of maintenance is NOT covered under warranty.

If defective components are

identified and require replacement, the equipment should be put out of service until repaired.

## **A** CAUTION

In order to maintain a safe level of operation, equipment must be inspected on a routine basis for damaged or worn parts. Failure to inspect equipment may result in injury to yourself or others.

### ISO1000R/7000R Maintenance Schedule

Any mechanical or electrical work within the main body of a medical CE class II unit MUST be recalibrated. The generic maintenance schedule below should be applied to medical CE and non-medical CE products.

COMPONENT	USE	WHEN	<b>BY WHOM</b>	
Covers, Seat, Handlebars and Console	Damp Cloth	Daily	Club Maintenance	
LCD Screen	LCD TV Screen Soft Cleaning Cloth	Weekly	Club Maintenance	
Chains (ISO7000R)	Lightly Applied Chain Lubricant	Quarterly	Club Maintenance	
Nuts and Bolts	Tighten When Neccessary	Bimonthly	Club Maintenance	
Base Roller Guide Track	Damp Cloth	Monthly	Club Maintenance	

### UNITED STATES CUSTOMER SERVICE

For assistance in the service of SCIFIT products;

phone :	(800) 745-1373 or (918) 359-2040
fax :	(918) 359-2045
e-mail:	service@scifit.com

The Product Support department is staffed from 7:00 AM to 6:00 PM CST Monday through Friday. A voice mail service is available 24 hours a day for recording messages to request technical support and to order replacement parts. Our mailing address is:

SCIFIT Systems Inc. 5151 S. 110th E. Ave. Tulsa, OK 74146 USA

### **UK & EUROPEAN CUSTOMER SERVICE**

phone :	+44 1344 300022	
fax:	+44 1344 868838	
e-mail:	info@scifit.uk.com	

SCIFIT LTD (UK) Lexham House Forest Road Binfield Berkshire, RG42 4HP UK

Company Number: 5970624 (UK)

### \*COUNTRIES OUTSIDE OF UK & EUROPE PLEASE USE USA CONTACT NUMBERS\*

Order online 24/7 at www.SCIFIT.com

### 20

### It is very important that your SCIFIT machine is registered. This can be done online at *http://www.SCIFIT.com/warrantyregistration.shtml* or fill out and mail the registration form at the back of this manual.

SCIFIT Statement of Warranty

SCIFIT warranties new products against defective workmanship and/or materials under normal and proper use subject to the following limitations:

(a) SCIFIT's obligation to the original purchaser shall apply to: Within the United States and Canada both parts and the cost of labor required to replace or repair a defective product for a period of one (1) year from *user/dealer purchase date as documented by \*warranty card and if warranty card has not been returned by user/dealer, then date of shipment from the factory.* Thereafter, for a period of two (2) years, such obligation shall extend only to the supply of replacement parts or products with any labor costs associated with such replacement or repair to be at Buyer's expense. Refer to clause (e) for components outside this policy clause.

Outside the United States and Canada replace defective product with no labor for a period of three (3) years from user purchase date as documented by \*warranty card and if warranty card has not been returned by user then date of shipment from factory.

\*Note: Original purchaser must register their purchased products either by warranty card return, web site registration or fax to activate warranty period or shipment date is extant for start of warranty period. A ninety (90) day period is to be given for warranty registration to allow stock rotation and showroom stock thereafter none registration will be shipment date for the start of the warranty period.

(b) SCIFIT's obligation shall be limited to repairing or replacing defective parts. No allowance shall be granted for repairs made by Buyer without SCIFIT's prior written approval. The decision to replace or repair shall be solely at SCIFIT's discretion.

(c) SCIFIT's warranty does not apply to parts requiring replacement or repair due to normal and abnormal wear and tear, improper use, corrosion (perspiration), improper maintenance, improper installation, improper rated, grounded or dedicated electrical circuits or improper storage, nor does it apply where all or part of the product has been altered from its original state by Buyer or a third party.

(d) THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, ARISING BY LAW OR OTHERWISE INCLUDING WARRANTY OR MERCHANTABILITY OF FITNESS FOR PARTICULAR PURPOSE, AND IS IN LIEU OF ALL OTHER LIABILITIES OF SCIFIT INCLUDING DIRECT, INDIRECT, SPECIAL AND CONSEQUENTIAL DAMAGES OR PENALTIES EXPRESSED OR IMPLIED WHETHER ARISING OUT OF CONTRACT, NEGLIGENCE OR OTHER SORT.

### SCIFIT STATEMENT OF WARRANTY

(e) The below listed items have the following warranty coverage unless determined to be defective. These items include, but are not limited to:

Treadmills – AC5000 models only	Warranty Period
Treadmill belts	12 months
Treadmill belts decks	12 months
Treadmill structure & frame	5 years
Treadmill drive system inclusive of motor & inverter	5 years
Treadmill Handrails & Handles	3 Years
Treadmill Heart rate grips	12 months
Treadmill Bottle Holder	90 days
Treadmill trays	90 days

Rotary Products	Warranty Period	
Upholstery	12 months	
Saddles/Seats	12 months	
Rotary Pedals	12 months	
Rubber grips	12 months	
Heart rate grips	12 months	
Rotary structure & Frame	5 years	
Rubber foot Pads (BioFlex)	12 months	
Rotary water bottle holders	90 days	
Rotary trays	90 days	

Accessories Products	Warranty Period
Pedal straps	Normal wear & tear
Heart Rate Receiver/Transmiter	90 Days
Low support boots	12 months
High support boots	12 months
Assist gloves	90 days
USB Keys and Lanyards	90 days

(f) Fires, floods, and acts of God, are not covered under this warranty.

#### **Freight and Shipping**

SCIFIT is not responsible for the repair or replacement of any unit or part damaged during transit or installation. <u>The customer is responsible for inspection of each unit and part for shipping damage at time of delivery or installation, and prior to signing receiving paperwork.</u> The customer is responsible for pursuing all freight damage claims with the appropriate transit company. If the customer signs an unqualified receipt for freight damage goods, the customer is solely responsible for the cost of the repair or replacement for such freight damage.

#### Maintenance

After training, always wipe down your SCIFIT exercise product. Perspiration that continuously settles on frame, upholstery, cas ings and control panels may eventually cause rust or damage. Damage resulting from lack of maintenance will NOT be covered under warranty. To clean upholstery, use mild soap and warm water. Dry with a clean towel. Refer to the Treadmill maintenance schedule.

#### **Product Support**

Assistance for the service of SCIFIT products is available by calling (800) 745-1373 or fax (918) 359-2045. The product support department is staffed from 7 AM to 5 PM Central Time Monday through Friday. A voicemail service is available 24 hours daily for recording messages to request technical support and to order replacement parts. Our goal is to return every voicemail call within 30 minutes of when it is placed during our normal business hours.

Please have the following information prior to calling technical support:

- Model number of equipment
- Serial number of equipment
- Point of contact name and phone number
- · Detailed description of symptoms encountered.

73

#### Installation

SCIFIT is not responsible for the repair or replacement of any unit or part damaged during installation. The customer is responsible for inspection of each unit and part for damage at the time of installation. The customer is responsible for pursuing all damage claims with the installer.

#### Software, Trademarks, Copyrights, and Patents

If an order includes software, such computer software is transferred by SCIFIT to the customer pursuant to a single user license, the royalty, terms, and conditions of that are set forth on or in the package accompanying such software.

KeyMaster software will receive 1 year of call-in technical support and owners will be eligible for product upgrades for one year following purchase.

SCIFIT has trademarked several names to uniquely identify its business and products. These names must not be used by other entities in the fitness business.

SCIFIT decals, user's manuals, and service manuals are copyrighted and may not be copied without prior approval from SCIFIT.

SCIFIT has obtained several patents on features and designs that are unique to its products. SCIFIT will defend these patents against those who attempt to utilize these features and designs in other products.

#### **Parts Shipment**

During the first 30 days warranty parts will be shipped via overnight delivery. Determination must be made before 2:00 PM Central Time on any given weekday for next day delivery. During the remainder of the first year warranty period, parts requirements will be filled via ground shipment. The customer is welcome to request overnight or 2nd day parts shipping, at customer's expense. If requested, SCIFIT will charge the customer's UPS account, or COD the difference in freight cost between ground shipment and overnight or 2nd day.

### **Return of Parts**

SCIFIT is committed to continual improvement in the equipment we market. In order to meet this commitment, the rapid return of defective parts is essential. The examination of the parts by our engineering department leads to changes that insure the same problem does not re-occur. Thank you in advance for your assistance!

When requested by SCIFIT, defective parts must be returned to the SCIFIT factory within 20 days of receipt of replacement part. Otherwise SCIFIT will expect payment on the parts invoice net 30 days.

### Please follow these three easy steps for returning parts.

### <u>Step 1</u>

Keep the box and packing material in which the new parts arrived.

### <u>Step 2</u>

Wrap the defective part and place in the box for safe return. Please take the brief moment needed to fill in the return parts form that is enclosed in the box with the new parts.

### Step 3

A **UPS prepaid Return Label** will be in the part box for only those parts that need to be returned. When **ALL** of the parts are received and inspected at the factory, a credit will be issued for the original parts invoice. **Attention service companies** - labor invoices will **NOT** be paid until defective parts are returned to the factory.

### Service Labor

Where applicable, the SCIFIT product support personnel will arrange a local field service technician to provide field support. Every effort will be made to schedule service during 48 business hours (8 hours per business day) following notification of a problem or as soon as repair parts are available to the field service technician. Where possible, parts will be supplied in advance of the field service technicians so that the product is repaired with one call.

#### **Purchased Parts**

All purchased parts will carry a 12 month warranty. Refer to Purchased parts shipments and installation for more details.

This Limited Commercial Warranty supercedes the limited commercial warranty printed in the "Users Operation Manual" for all SCIFIT Systems, Inc. products.

If you have questions or require additional information, please contact SCIFIT Systems, Inc. at 1-800-745-1373 or service@scifit.com



### SCIFIT<sup>®</sup> arranty Car



SCIFIT 5151 South 110th East Avenue Tulsa, OK 74146

From

Important: Please keep this page for "Recalibration records"	
Model Number:	
Serial Number:	
Date of Purchase:	
Calibration Date (Medical units):	
Supplied By: SCIFIT Systems Inc.	
User Manual: #P4642A (11/2010)	

1

I

I

1

I

I

	DATE	CLUB OR CERTIFICATION LOCATION	CERTIFIED SERVICE COMPANY	TECH INITIALS
1st Recalibration				
2nd Recalibration				
3rd Recalibration				
4th Recalibration				
5th Recalibration				

Please make extra blank chart copies for future use



## Scientific Solutions for Fitness.

5151 South 110th East Avenue Tulsa, Oklahoma 74146 USA

Sales: 1-800-278-3933 • 1-918-359-2000 Customer Service: 1-800-745-1373 www.scifit.com

Order online 24/7at www.SCIFIT.com