Models: Latitude-xxx01



Prescribed for Progress™

Latitude[™] User's Machine Manual



Lateral StabilityTrainer



Before Using this product, read this manual and follow all safety rules and operating instructions.



CORPORATE HEADQUARTERS

Columbia Centre III, 9525 West Bryn Mawr Avenue, Rosemont, Illinois 60018 • U.S.A. 847.288.3300 • FAX: 847.288.3703

Service phone number: 800.351.3737 (toll-free within U.S.A., Canada)

Global Websitewww.lifefitness.com

INTERNATIONAL OFFICES

AMERICAS

North America SCIFIT

5151 S. 110th E. Avenue Tulsa, OK, 74146 U.S.A Toll Free: +1 (800) 745 1373 Fax: +1 (918) 359 2045 Email: Service@scifit.com

Service Website: www.scifit.com/Support Operating Hours: 7:00 am - 5:00 pm (CST)

Brazil Life Fitness Brasil

Av. Cidade Jardim, 900 Jd. Paulistano São Paulo, SP 01454-000 BRAZIL SAC: 0800 773 8282

Telephone: +55 (11) 3095 5200 Fax: +55 (11) 3095 5201

Service Email: suporte@lifefitness.com.br Sales/Marketing Email: lifefitness@lifefitness.com.br

Service Operating Hours: 9:00 - 17:00 (BRT) (Monday-Friday)

Store Operating Hours: 9:00 - 20:00 (BRT) (Monday-Friday) 10:00 - 16:00 (BRT) (Saturday)

Latin America & Caribbean* Life Fitness Inc.

Columbia Centre III 9525 West Bryn Mawr Avenue Rosemont, IL 60018 U.S.A Telephone: (847) 288 3300 Fax: (847) 288 3703 Service Email:

customersupport@lifefitness.com Sales/Marketing Email:

commercialsales@lifefitness.com
Operating Hours: 7:00am - 6:00pm (CST)

EUROPE, MIDDLE EAST, & AFRICA (EMEA)

Netherlands & Luxemburg Life Fitness Atlantic BV

Bijdorpplein 25-31 2992 LB Barendrecht THE NETHERLANDS Telephone: (+31) 180 646 666 Fax: (+31) 180 646 699 Service Email:

service.benelux@lifefitness.com Sales/Marketing Email:

Marketing.benelux@ifefitness.com Operating Hours: 9.00h - 17.00h (CET)

United Kingdom & Ireland Life Fitness UK LTD

Queen Adelaide Ely, Cambs, CB7 4UB Telephone: General Office (+44) 1353.666017 Customer Support (+44) 1353.665507 Fax: (+44) 1353.666018 Service Email: uk.support@lifefitness.com

Sales/Marketing Email: life@lifefitness.com
Operating Hours:
General Office: 9.00am - 5.00pm (GMT)
Customer Support: 8.30am - 5.00pm

GMT)

Germany & Switzerland Life Fitness Europe GMBH Siemensstraße 3

85716 Unterschleißheim GERMANY Telephone: (+49) 89.31 77 51.0 (Germany) (+41) 0848 000 901 (Switzerland) Fax: (+49) 89.31 77 51.99 (Germany) (+41) 043 818 07 20 (Switzerland) Service Email: kundendienst@lifefitness.com Sales/Marketing Email: kundenberatung@lifefitness.com Operating Hours: 08.30 - 16.30h (CET)

Austria Life Fitness Austria

Vertriebs G.m.b.H. Dückegasse 7-9/3/36 1220 Vienna AUSTRIA Telephone: (+43) 1.61.57.198 Fax: (+43) 1.61.57.198.20 Service Email: kundendienst@lifefitness. com

Marketing/Sales Email: kundenberatung@lifefitness.com Operating Hours: 08:30 - 16.30.h (MEZ)

Spain Life Fitness IBERIA

C/Frederic Mompou 5,1°1a

SPAIN
Telephone: (+34) 93.672.4660
Fax: (+34) 93.672.4670
Service Email: servicio.tecnico@lifefitness.com
Sales/Marketing Email:
info.iberia@lifefitness.com
Operating Hours:
9.00h - 18.00h (Monday-Thursday)
8.30h - 15.00h (Friday)

08960 Sant Just Desvern Barcelona

Belgium Life Fitness Benelux NV

Parc Industrial de Petit-Rechain

4800 Verviers
BELGIUM
Telephone: (+32) 87 300 942
Fax: (+32) 87 300 943
Service Email:
service.benelux@lifefitness.com
Sales/Marketing Email:
marketing.benelux@lifefitness.com
Operating Hours: 9.00h - 17.00h (CET)

All Other EMEA Countries & Distributor Business C-EMEA*

Bijdorpplein 25-31 2992 LB Barendrecht THE NETHERLANDS Telephone: (+31) 180 646 644 Fax: (+31) 180 646 699 Service Email: service.db.cemea@lifefitness.com Sales/Marketing Email:

marketing.db.cemea@lifefitness.com Operating Hours: 9.00h - 17.00h (CET)

ASIA PACIFIC (AP)

Japan Life Fitness Japan

Japan Nippon Brunswick Bldg., #8F 5-27-7 Sendagaya Shibuya-Ku, Tokyo Japan 151-0051 Telephone: (+81) 3.3359.4309 Fax: (+81) 3.3359.4307 Service Emaiservice@lifefitnessjapan.com Sales/Marketing Email: sales@lifefitnessjapan.com Operating Hours: 9.00h - 17.00h (JAPAN)

Hong Kong Life Fitness Asia Pacific LTD

32/F, Global Trade Square
21 Wong Chuk Hang Road
HONG KONG
Telephone: (+852) 2575.6262
Fax: (+852) 2575.6894
Service Email:
HongKongEnquiry@lifefitness.com
Sales/Marketing Email:
Marketing.HK.Asia@lifefitness.com
Operating Hours: 9.00h - 17.00h
Service Operating Hours: 9.00h - 18.00h

All Other Asia Pacific countries & distributor business Asia Pacific Life Fitness Asia Pacific LTD

32/F, Global Trade Square
21 Wong Chuk Hang Road
HONG KONG
Telephone: (+852) 2575.6262
Fax: (+852) 2575.6894
Service Email:
HongKongEnquiry@lifefitness.com
Sales/Marketing Email:
Marketing.HK.Asia@lifefitness.com
Operating Hours: 9.00h - 17.00h
Service Operating Hours: 9.00h - 18.00h

^{*} Also check www.lifefitness.com for local representation or distributor/dealer

^{*} Order online 24/7 at www.SCIFIT.com



Prescribed for Progress™

Operation Manuals and other Product Information available at http://www.scifit.com/resources/specification-manuals/

http://www.scifit.com/resources/specification-manuals/

操作マニュアルおよびその他の製品情報については、http://www.scifit.com/resources/specification-manuals/をご覧ください。

如需操作手册和其他产品信息,请登录: http://www.scifit.com/resources/specification-manuals/

操作手冊和其他產品資訊請見 http://www.scifit.com/resources/specification-manuals/

작동 설명서 및 기타 제품 정보는 http://www.scifit.com/resources/specification-manuals/ 에서 찾을 수 있습니다.

Trobareu el manual de funcionament i altra informació de producte a http://www.scifit.com/resources/specification-manuals/

Mae Llawlyfrau Gweithredu a Gwybodaeth Arall am Beiriannau ar gael yn http://www.scifit.com/resources/specification-manuals/

Die Betriebsanleitung und andere Produktinformationen erhalten Sie unter http://www.scifit.com/resources/specification-manuals/

Encontrará el manual de funcionamiento y otra información de producto en http://www.scifit.com/resources/specification-manuals/

Erabiltzailearen Eskuliburua eta Beste Produktuei buruzko Informazioa eskuratu http://www.scifit.com/resources/specification-manual

Käyttöoppaat ja muut tuotetiedot ovat saatavana osoitteessa http://www.scifit.com/resources/specification-manuals/

Les manuels d'utilisation, ainsi que d'autres informations sur les Produits, sont disponibles sur http://www.scifit.com/resources/specification-manuals/

Felhasználói kézikönyvek és más termékinformációk: http://www.scifit.com/resources/specification-manuals/

Manuali di funzionamento e altre informazioni sui prodotti disponibili su http://www.scifit.com/resources/specification-manuals/

Manuali di funzionamento e altre informazioni sui prodotti disponibili su http://www.scifit.com/resources/specification-manuals/

Instrukcje obsługi i inne informacje o produktach dostępne na stronie http://www.scifit.com/resources/specification-manuals/

Manuais de Operação e Informações sobre outros produtos disponíveis em http://www.scifit.com/resources/specification-manuals/

Руководства по эксплуатации и другую касающуюся нашей продукции информацию можно найти, перейдя по ссылке http://www.scifit.com/resources/specification-manuals/

Kullanım Kılavuzları ve Diğer Ürün Bilgileri http://www.scifit.com/resources/specification-manuals/

TABLE OF CONTENTS

INTRODUCTION	1
MAINTENANCE	1
SAFETY INFORMATION	2
SPECIFICATIONS	3
SAFETY PERIMETER	3
CONSOLE ATTACHMENT	4
SEAT ASSEMBLY MOUNTING	5
MACHINE RELOCATION	5-6
BIOMECHANICAL POSITIONING	7
WARRANTY	8

INTRODUCTION

Thank you for your purchase of the SCIFIT Latitude Lateral Stability Trainer. We have incorporated the finest technology and ergonomic design into this machine to assist you in achieving your fitness goals.

The SCIFIT Latitude Lateral Stability Trainer is designed for class "S" (commercial) and class "A" (high accuracy) applications. The intended uses are for general health and wellness, cardio & pulmonary rehabilitation, sports medicine, physical therapy and physical conditioning. SCIFIT equipment is suitable for placement in medical, wellness, fitness, and recreational facilities. The Latitude Lateral Stability Trainer has 191 levels of console controlled resistance, which ensures that users of all fitness levels will be able to get a challenging but not overwhelming workout.

To review the latest guidelines and recommendations, visit the American College of Sports Medicine website: www.acsm.org

FCC Warning - Possible Radio / Television Interference

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

MAINTENANCE

COMPONENT	ACTION	WHEN
Upholstery	Clean With Mild Soap and Water	Daily
Base Roller Track	Clean With a Damp Cloth	Weekly
Battery	Refer to "Console Prog. Manual" For Battery Check Instructions	Monthly

AATTENTION

Contact your local waste management agency for specific requirements concerning recycling and disposal locations of batteries.

SAFETY INFORMATION

The purpose of the symbols seen below are to attract your attention to possible dangers during the assembly or adjustment process. The Attention, Warning symbols and special icons with its associated explanation, deserves your careful consideration. The warning does not by itself eliminate any danger. The instructions given are not substitutes for proper accident prevention measures.

Look for these symbols that will point out safety precautions or important information.









- Read all warnings and instructions on each product prior to using your SCIFIT exercise machine. Save these instructions for future reference.
- If warnings are missing or damaged, please contact SCIFIT immediately for replacement warning labels. Warning labels are shipped with every product and should be installed before product is used. SCIFIT is not responsible for missing or damaged warning labels.
- Use the exercise machine only for its intended purposes as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never drop or insert any object into any opening.
- This machine is intended for indoor use only. Do not place in direct sunlight, high humidity or where water is present (i.e. pool or spa areas).
- Consult your physician or medical specialist before participating in any exercise program, especially if you are pregnant or if you are suffering from heart disease, respiratory disease, diabetes, high blood pressure, elevated cholesterol, arthritis, or any other diseases and physical complaints.
- The exercise machine is not to be used by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have supervision or been given instruction.
- Close supervision is necessary when the exercise machine is being used near children. Keep children away from rotating mechanical parts and supports that may cause injury.
- To prevent muscle injuries and soreness, you should always cool down at least 5 minutes. Always stretch after a workout session to reduce the risk of personal injury. Failure to stretch properly may result in injury to yourself.
- WARNING! Heart rate monitoring systems may be inaccurate. Over exercise may result in serious injury or death If you feel faint stop exercising immediately.
- **DANGER!** To reduce the risk of electrical shock, always unplug SCIFIT products before cleaning or attempting any maintenance activity.
- **WARNING!** To reduce the risk of burns, fire, electrical shock, or injury, it is imperative to connect each product to a properly grounded electrical outlet.
- Injuries to health may result from incorrect or excessive training. Failure to stop exercising can result in injury or death to yourself.

PROPOSITION 65

Health and Environmental Regulation Warning - This product may contain chemicals known to the state of California to cause cancer, birth defects, or other reproductive harm. For more information related to the European Commission Regulation (EC) No. 1907/2006 (REACH) and the California Safe Drinking Water and Toxic Enforcement Act of 1986 (Proposition 65), please visit https://www.lftechsupport.com/web/guest/environment-regulations-information.

SPECIFICATIONS Max User Weight......Standard Seat - 500 lbs (227 kg) Bariatric Seat - 600 lbs (272 kg) Premium Seat - 450 lbs (204 kg) Resistance System......Electromagnetic Self Generating Resistance ProgramsQuick Start, Manual, Constant Workout, Hills, Heart Rate, Random, Heart Fit Test and Heart Fit Training Accuracy Class Class A Machine Weight: 322lbs (146 kg) **Length:** 72 in. (183cm) Intelli-Fit **Height:** 49.5 in. (126cm) Console Width: 34 in. (88cm) **Ergonomic Seat Assembly** (Premium Seat Shown) **Oversized Comfortable Footbeds Foot Lever For** Forward/Backward **Seat Adjustment** Leveling **Endcaps SAFETY PERIMETER** 40" (1m) 40" (1m) **14444444** 00000000 40" (1m) 40" (1m)

CONSOLE ATTACHMENT

Required Tools: #2 Phillips Screwdriver



No Power Tool

Components:




Hardware:






Neck Weldment and Comm. Cables (HUR)



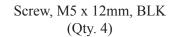





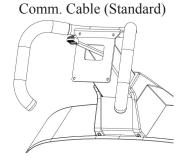







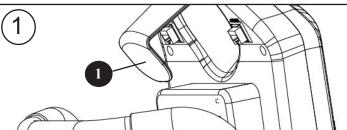


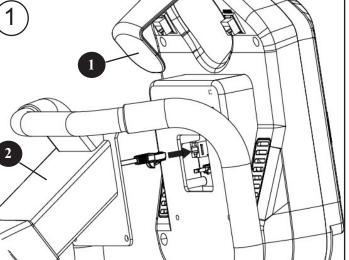


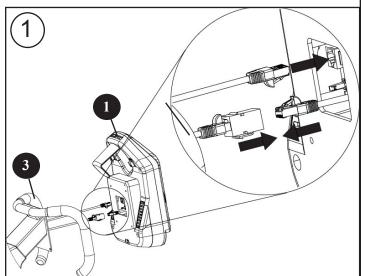

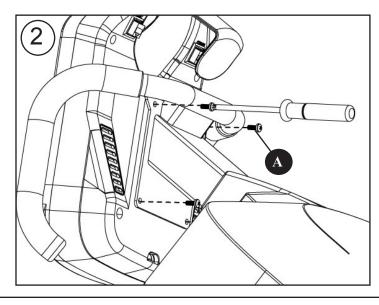

SEAT ASSEMBLY MOUNTING

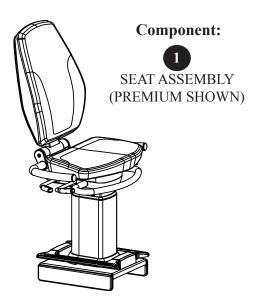

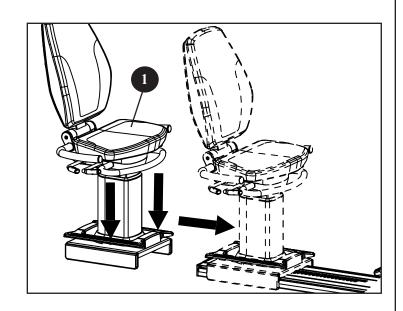

MACHINE RELOCATION





A WARNING


Failure to place the machine on a stable and level surface may result in injury to yourself and others.

SEAT ASSEMBLY (PREMIUM SHOWN)




RH/LH OFFSET ENDCAPS


Components:

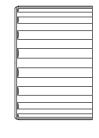





MACHINE RELOCATION

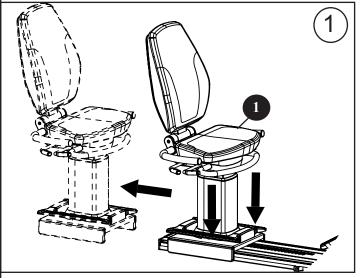

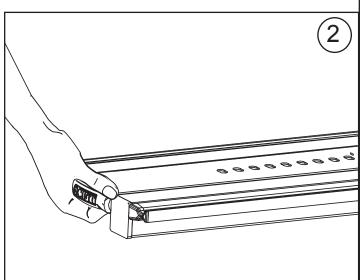

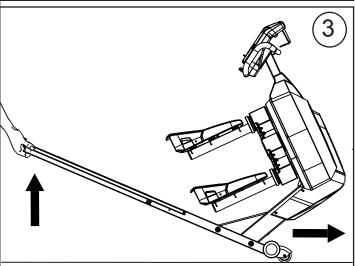

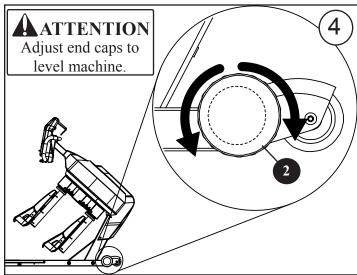

NOTES

BIOMECHANICAL POSITIONING

1. MACHINE ENTRY:

(a) Press and hold the yellow foot lever located on either side of the seat base (see Seat System manual), then move the seat back to allow sufficient entry space and release the foot lever. You may grab the handles on either side of the console to stabilize yourself and sit on the seat.

OR

Lift upward on the yellow swivel lever (Premium only) located on either side of the seat (see Seat System manual) and rotate the seat 90 degrees to the right or left to access the bottom seat pad and release, then sit down and again lift upward on the swivel lever and rotate back into the original position.

- (b). Position the seat forward or backward to a comfortable location using the yellow foot lever, then release the lever allowing for a slight knee bend when placing your feet on the foot pads.
- (c). Adjust the height (Premium only) of the seat using the yellow bar on the front of the seat (see Seat System manual) to obtain a comfortable position.

2. BEGIN A WORKOUT:

- (a) Begin and continue rotating the foot pads to turn on the console.

 NOTE: The AC adapter may also be connected to the machine to keep the console continuously powered-up.
- (b). Select the workout program preferred (see Console Program manual) and begin the workout.
- (c). You may stabilize yourself during the workout by grabbing the seat handlebars located on either side of the bottom seat pad.

3. EXITING THE MACHINE WHEN THE WORKOUT HAS COMPLETED:

(a) Press and hold the yellow foot lever located on either side of the seat base, then move the seat back to allow sufficient exit space and release the foot lever. You may grab the handles on either side of the console for stabilization, then lift yourself and carefully exit the machine.

OR

Lift upward on the yellow swivel lever (Premium only) located on either side of the seat and rotate the seat 90 degrees right or left, then release. Carefully lift yourself upward and exit the machine.

WARRANTY

WARRANTY INFORMATION

WHAT IS COVERED.

This SCIFIT commercial exercise equipment ("Product") is warranted to be free of all defects in material and workmanship.

WHO IS COVERED.

The original purchaser or any person receiving a newly purchased Product as a gift from the original purchaser. Warranty will be voided on subsequent transfers.

HOW LONG IS IT COVERED.

All coverage is provided by specific Product according to the guidelines READ THE MANUAL before operating the Product. Remember to listed on the chart below.

WHO PAYS TRANSPORTATION & INSURANCE FOR SERVICE.

If the Product or any covered part must be returned to a service facility for repairs, We, SCIFIT, will pay all transportation and insurance information. Reference your name, address and the serial number of charges for the first year. You are responsible for transportation and insurance charges during the second and third years (if applicable).

WHAT WE WILL DO TO CORRECT COVERED DEFECTS.

We will ship to you any new or rebuilt replacement part or component, or, at our option, replace the Product. Such replacement parts are warranted for the remaining portion of the original warranty period.

WHAT IS NOT COVERED.

Any failures or damage caused by unauthorized service. misuse, accident, negligence, improper assembly or installation, debris resulting from any construction activities in the Product's environment, rust or corrosion as a result of the Product's location. alterations or modifications without our written authorization or by failure on your part to use, operate and maintain the Product as set out in your Operation Manual ("Manual").

One type RJ45 interchangeable Network ready and Fitness Entertainment port is supplied with the Product. This port complies with the FitLinxx CSAFE specification dated August 4, 2004 stating: 4.75VDC to 10VDC; maximum current of 85mA. Any Product damage caused by a load exceeding this FitLinxx CSAFE specification is not covered by warranty.

All terms of this warranty are void if this product is moved beyond the continental borders of the United States of America (excluding Alaska, Hawaii and Canada) and are then subject to the terms provided by that country's local authorized SCIFIT representative.

Detailed warranty guidelines are available at: http://www.SCIFIT.com/Support/Service information/Warranty and Product Information/

WHAT YOU MUST DO.

Register your Product online at

http:/www.SCIFIT.com/warrantyregistration.shtml/. Retain proof of purchase; use, operate and maintain the Product as specified in the Manual; notify Customer Support Services of any defect within 10 days after discovery of the defect; if instructed, return any defective part for replacement or, if necessary, the entire Product for repair.

OPERATION MANUAL: It is VERY IMPORTANT THAT YOU perform the periodic maintenance requirements specified in the Manual to assure proper operation and your continued satisfaction.

HOW TO GET REPLACEMENT PARTS & SERVICE.

Refer to page one of this manual for your local service contact your Product. They will tell you how to get a replacement part, or, if necessary, arrange for service where your Product is located.

EXCLUSIVE WARRANTY.

THIS LIMITED WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES OF ANY KIND EITHER EXPRESSED OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRAN-TIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND ALL OTHER OBLIGATIONS OR LIABILITIES ON OUR PART. We neither assume nor authorize any person to assure for us any other obligation or liability concerning the sale of this Product. Under no circumstances shall we be liable under this warranty, or otherwise, of any damage to any person or property, including any lost profits or lost savings, for any special, indirect, secondary, incidental or consequential damages of any nature arising out of the use of or inability to use this Product. Some states do not allow the exclusion or limitation of implied warranties or of liability for incidental or consequential damages, so the above limitations or exclusions may not apply to you. Warranty coverages and terms may differ outside the United States. Please contact the SCIFIT office servicing your country (contact information found at the front of this manual) or visit the applicable local SCIFIT website to receive the specific warranty information for your country.

CHANGES IN WARRANTY NOT AUTHORIZED.

No one is authorized to change, modify or extend the terms of this limited warranty.

EFFECT OF STATE LAWS.

This warranty gives you specific legal rights and you may have other rights which vary from state to state and country by country.

OUR PLEDGE TO YOU.

Our Products are designed and manufactured to the highest standards. We want you completely satisfied with our Products and will do everything possible under the terms of this warranty to keep you secure in knowing you have bought the best!

Item	6 mo.	1 Year	2 Years	3 Years	5 Years
Frame					X
Electrical				X	
Mechanical (except cranks & drive system)				X	
Labor		X			
Cranks/Drive Syst. (Latitude only)			X		
Upholstery, Seats, Grips, Walk					
belts, Decks		X			
Battery		X			
High Wear Items	·				
(Heartrate Straps, Pedal Straps)	X				

